

## Synthetic Marijuana Test

The **LaneWorkSafe Synthetic Marijuana Test** is intended to be used in the screening of a urine specimen for the presence of Synthetic Cannabis and its metabolites. The **LaneWorkSafe Synthetic Marijuana Test** detects for a total of **10 compounds** found in the substance making it the most accurate and reliable device available.

### Two most common ingredients:

JWH018	N-pentanoic acid metabolite
JWH073	Butanomic acid metabolite

★★★★  
**TESTS for 10  
COMPOUNDS**

### Features

- User friendly
- Accurate
- Quick! Results in 5-7 minutes
- Uses alternate immunochromatographic technology

<b>JWH-018</b>	<b>JWH-073</b>
<b>JWH-018-N-4</b>	<b>JWH-073-N-4</b>
<b>JWH-081-N-5</b>	<b>JWH-200-N-6</b>
<b>AM-2201-N-4</b>	<b>JWH-250-N-5</b>
<b>RCS-4-N-5</b>	<b>LAMOTRIGINE</b>



### 3 SIMPLE STEPS

1

Remove test from pouch. Write donor name/ID on plastic. Remove cap.

2

Immerse tip into urine specimen for 30 seconds. Replace cap and lay flat

3

Read results in 5-7 minutes.