

# Split-Specimen Cup™ Procedure

## 1 Collect specimen




Verify temperature

## 2 Insert key




## 3 Peel back label

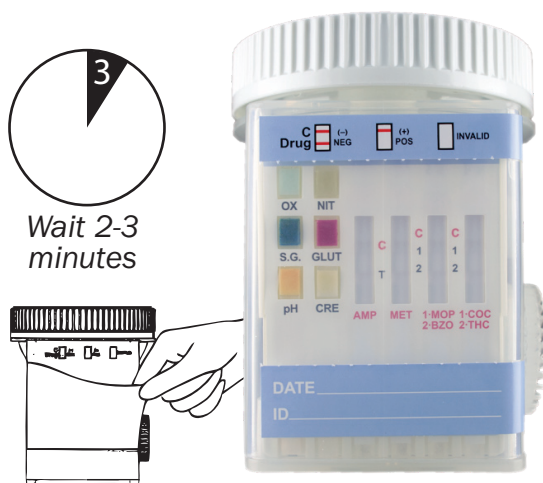

Wait 2-3 minutes

## 4 Read the results

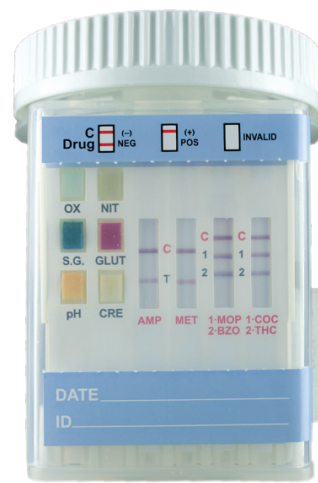

Drug **NEGATIVE**  
 Drug **POSITIVE**  
 Drug **INVALID**

All positive results are presumptive and should be confirmed by an alternate method. (eg: GC/MS)

### ADULTERANTS LEGEND CARD:

Abnormal	Normal	Normal	Abnormal
OX PCC Low High	Negative 1.003 1.005 1.015 1.025	20mg/dl Negative 100mg/dl	NIT 50mg/dl Low 100mg/dl High
S.G. 1.000 Low ≥1.035 High	4.0 7.0 9.0	Negative 100mg/dl 20mg/dl	GLUT Low High
pH 2.0 3.0 10.0 11.0 12.0 Low High		0mg/dl Low 10mg/dl High	CRE

### ADULTERANTS:

Oxidants (Pyridinium Chlorochromate)  
Specific Gravity  
pH

Nitrite  
Glutaraldehyde  
Creatinine

### SUBSTANCES

Drug Name:	Abbreviation:	Cut-off Levels:
Amphetamine	AMP	300ng/mL
Methamphetamine	mAMP	300ng/mL
Cocaine	COC	300ng/mL
Marijuana	THC	50ng/mL
Benzodiazepines	BZO	200ng/mL
Morphine	MOP	300ng/mL