

# Quantisal™ Oral Fluid Collection Device

Easy, non-invasive, observed collection

The Quantisal™ collection device is recognized as a reliable and innovative testing format that offers fast and easy collection of oral fluid in order to detect the presence of drugs in adults 18 years and older.




## Key Benefits


- Volume adequacy indicator
- Collects 1 mL ± 10% oral fluid
- Excellent extraction efficiencies
- No artificial stimulants

## How to Collect a Specimen with Quantisal


- 1** Check expiration date on Quantisal packaging and ensure donor has refrained from consumption of food or beverage for 10 minutes prior to specimen collection.

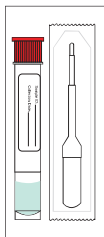


- 2** Instruct donor to peel open package and remove collector. To expedite the collection process, **have donor move tongue side to side to accumulate saliva** in mouth before starting. Keep the tip of the device pointed down.

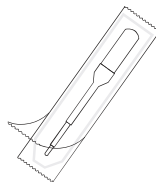

- 3** Instruct donor to position collector under tongue and close mouth. **Keep head down to allow gravity to help with saliva collection.**  
**IMPORTANT:** ensure donor does not chew on pad, talk, or remove collector from mouth until indicator turns **BLUE**, or until 10 minutes have passed.

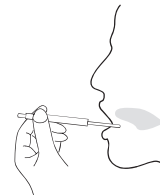

- 4** Instruct donor to hold transport tube in an upright position and uncapped by pushing up with thumb(s). Do not stand tube on table. Do not spill or empty the liquid from tube.

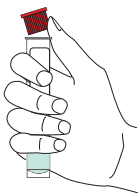

- 5** Instruct donor to insert collector into the uncapped transport tube and replace cap.



- 6** **SNAP CAP** firmly for transport. Place center of specimen seal on top of tube and press down both sides. Complete paperwork and send sample to laboratory.

